

PowerLine™ Central Venous Catheter

Small Diameter Power-Injection CVC



Specially indicated for the power injection of contrast media for CT scans, the PowerLine™ Central Venous Catheter is ideal for central venous pressure monitoring and administration of I.V. fluids, blood products, medications, and blood withdrawals.¹

Power Injectable¹

- Maximum 5 ml/sec flow rate at 300 psi pressure limit enables injection of contrast media for contrast enhanced computed tomography (CECT) scans

Versatility, Strength and Durability

- Single, dual, and triple lumen options help increase the flexibility of your treatment
- Indicated for central venous pressure monitoring¹
- Polyurethane construction adds strength and durability for both short-term and long-term use

Clear Labeling

- Easy to identify purple power injectable lumen
- Depth markings every 1 cm along catheter

The Power of Purple™


Robust luer connectors designed for durability during long-term use

Purple power injectable lumen labeled for easy identification

All extension legs labeled "Cuff" to identify as a tunneled catheter

I.D. tag indicates maximum injection rate



Compatible with Statlock™ CV Plus Stabilization Device¹

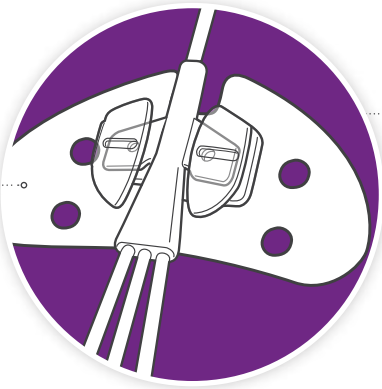

SURECUFF™ in-growth cuff promotes tissue growth to secure catheter placement

1 cm depth markings help placement accuracy

Single, dual, and triple lumen catheter configurations

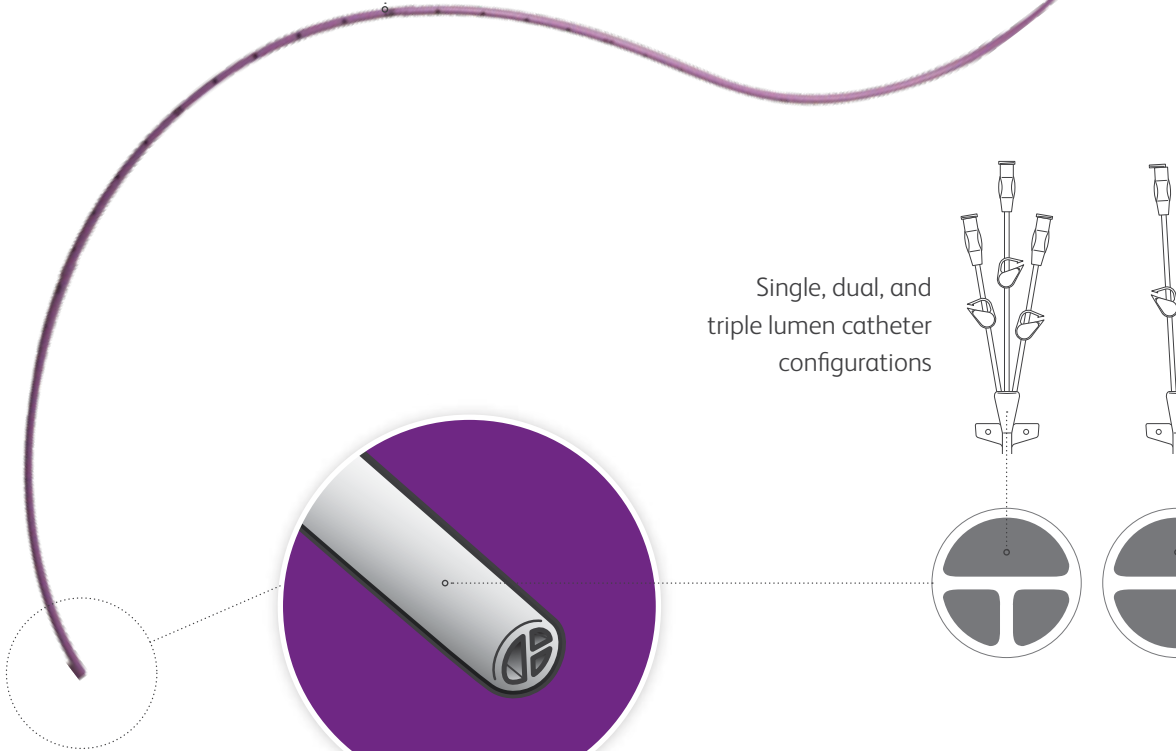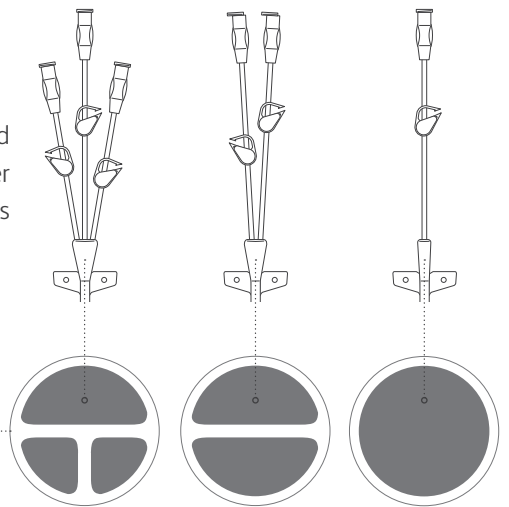

Ordering Information

Lumens	French Size	Tray	Priming Volume (MI)	Average Gravity Flow Rate (mL/hr)	Cuff Location (cm)	Length* (cm)	Overall Length (cm)	Product Code
Single	5F	Microintroducer	0.62	1245	5	50	62.4	6700515
Dual	5F	Microintroducer	0.55 / 0.55	577 / 577	5	50	63.3	6720515
Triple	6F	Microintroducer	0.67 / 0.45 / 0.45	924 / 360 / 360	5	50	65.8	6730615

* Distance from the most proximal measurement marker to catheter tip.

Tray Components

Microintroducer

- Reverse Taper Polyurethane Catheter with SURECUFF™ Tissue Ingrowth Cuff
- MICROEZ™ Microintroducer
- Hydro-Glide™ Insertion Stylet
- 21 G Introducer Needle
- 70 cm Guidewire (0.018 in.)
- Tunneler
- Scalpel
- Attachable Suture Wing
- Dressing, Transparent
- Injection Cap(s)
- Syringe
- Statlock™ Securement Device

Reference 1. PowerLine™ Central Venous Catheter Instructions For Use.

Please consult product labels and instructions for use for all indications, contraindications, Hazards and warnings and precautions.

벡톤디킨슨코리아(주)
서울특별시 강남구 테헤란로 142, 16층

bd.com

